

Infant Resuscitation Masks



SINGLE USE INFANT RESUSCITATION MASKS

Fisher & Paykel Healthcare Infant Resuscitation Masks are specifically designed to conform comfortably to an infant's face, facilitating a seal for optimal resuscitation.

The complete range of masks are designed for use with the Neopuff™ Infant T-Piece Resuscitator and can connect to other resuscitation devices.

Infant Resuscitation Masks | FEATURES

The Fisher & Paykel Healthcare Infant Resuscitation Masks are available in a range of sizes providing an anatomical seal.

Range Of Sizes

The infant resuscitation masks are available in five sizes. The range includes extra small sizes for micro-premature neonates to larger sizes for term infants and pediatric patients. This ensures there is a match for a wider patient range. A mask sizing chart is available in the starter kit to assist with mask selection for each patient.

Anatomical Seal

The extended edge of the mask is soft and pliable, easily creating a comfortable anatomical seal around the infant's mouth and nose. To achieve a good seal, it is recommended to gently roll the mask onto the face from the chin and to apply the C-Hold hand positioning.



1. Anatomical Seal

- Soft and pliable extended rim
- Firm sides provide support



2. Visibility

- Transparent surface
- Allows clear observation of breath condensate



3. Versatile Connection

- 15 mm conical male ISO 5356-1:2004
- Connects to common resuscitation devices



4. Range of Sizes

- Extra small mask sizes suitable for premature neonates
- 35 mm pictured left at actual size

INFANT RESUSCITATION MASKS

	RD803-10 Micro-Premature (XS)	RD804-10 Premature (S)	RD805-10 Neonatal (M)	RD806-10 Infant (L)	RD807-10 Pediatric (XL)
Diameter	35 mm (1.38")	42 mm (1.65")	50 mm (1.97")	60 mm (2.36")	72 mm (2.83")
Suggested Weight Range	400 g - 1 kg (2.3 lb)	<1.5 kg (3.3 lb)	<2.5 kg (5.5 lb)	<5 kg (11 lb)	<10 kg (22 lb)
	Extremely Low Birth Weight (ELBW)	Very Low Birth Weight (VLBW)	Low Birth Weight (LBW)	Term	
Material	Thermoplastic Elastomer (The product does not contain any latex or DEHP plasticizer).				
Cleaning and Disposal	Single patient use - supplied clean, not sterile. Discard according to hospital protocol.				

There is a range of kit options for the Infant Resuscitation Masks, including a Mask Starter Kit (RD900-EN). These options are included in the Neopuff Product Catalog.